## 1-Qt., 1.5-Qt., 1, 2 & 3-Gallon Sharps Recovery System™

### SYSTEM COMPONENTS (Save for packaging and return mailing.)


- Prepaid return shipping box with pre-attached USPS mailing label (return postage is prepaid)
- Inner box and 4 mil plastic bag liner
- Red sharps collection containers with absorbent pad inside
- Tape, twist-tie
- Tracking document on outside of shipping box
- Instructions


### INSTRUCTIONS (Read instructions including all warnings completely before using this system.)

**Fig. 1:** Remove container from packaging and place at point of use. **DO NOT REMOVE ABSORBENT PAD.**

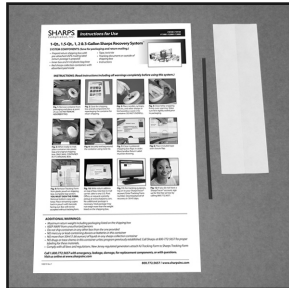**Fig. 2:** Save the shipping box and all components for repackaging the container for return shipping.

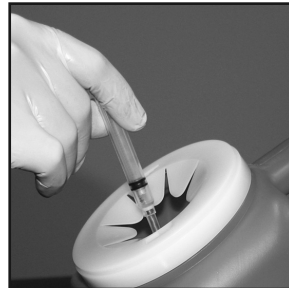
**Fig. 3:** Place needles, syringes, lancets, and other sharps or biohazardous waste into container. **DO NOT OVERFILL.**


**Fig. 4:** Close lid by snapping firmly over opening. Make sure lid is tightly sealed prior to packaging.

**Fig. 5:** When ready to mail, place container into plastic bag and original shipping box. **ONLY MAIL CONTAINER IN ITS ORIGINAL BOX.**

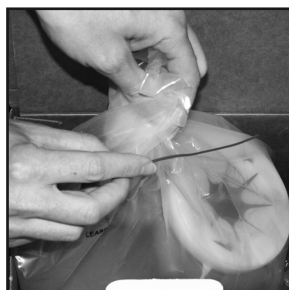**Fig. 6:** Securely seal bag around the container using twist-tie.

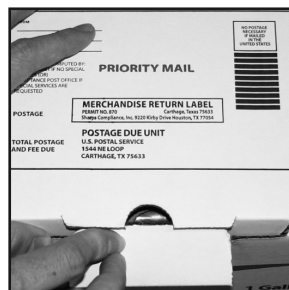
**Fig. 7:** Close numbered shipping box flaps in order. Merchandise Return Label must be showing.

**Fig. 8:** Place included tape over closure flap.


**Fig. 9:** Remove Tracking Form from plastic pouch on shipping box. Complete top section. **YOU MUST SIGN THE FORM.** Remove bottom copy and keep. Place remaining copies back in pouch with barcode facing out. Box will not be accepted without tracking form.

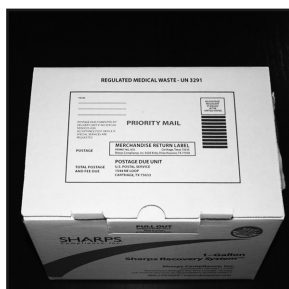**Fig. 10:** Write return address on top of box. Give box to mail carrier, take to any U.S. Post Office, or request a priority pickup at [www.sharpsinc.com](http://www.sharpsinc.com). No additional postage is necessary. Total package may not weigh more than the weight listed on the shipping box.

**Fig. 11:** For tracking purposes, log on to your SharpsTracer® account. Enter Tracking Form number. Download proof of recovery in 30-45 days.

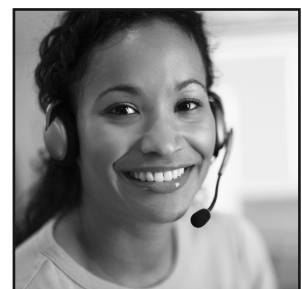**Fig. 12:** If you do not have a SharpsTracer® account, sign up for this free service by calling 800.772.5657.

### ADDITIONAL WARNINGS:

- Maximum return weight including packaging listed on the shipping box
- **KEEP AWAY** from unauthorized persons
- Do not ship containers in any other box than the one provided
- **NO** mercury or lead-containing devices or batteries in this container
- **NO** more than 50ml (1.66 ounces) of liquids in any sharps collection container
- **NO** drugs or trace chemo in this container unless program previously established. Call Sharps at 800-772-5657 for proper labeling for these materials.
- Comply with all laws and regulations. New Jersey regulated generators attach NJ Tracking Form to Sharps Tracking Form

**Call 1.800.772.5657 with emergency, leakage, damage, for replacement components, or with questions. Visit us online at [www.sharpsinc.com](http://www.sharpsinc.com)**

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