


Please check with facilities management or appropriate personnel before mounting the sharps container bracket.

Identify the location for the wall mount bracket which will be installed. In a clinical/healthcare setting, wall-mounted sharps containers should be installed 52-56 inches above the floor. When mounting in a public access area, mount the enclosure no higher than 48 inches above the floor with unobstructed access in compliance with the Americans with Disabilities Act.



- Ensure that the bracket is level on the wall before marking the screw holes
- Mark the wall through the holes located on the back of the bracket
- Apply wall anchors if needed
- Apply screws to the bracket to secure it to the wall surface

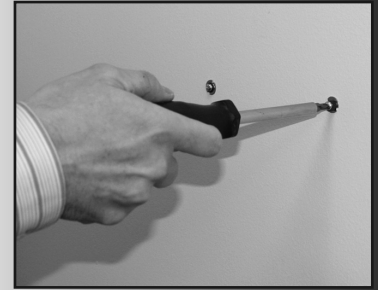

Model #50004: For use with 1-gallon, 2-gallon and 3-gallon Sharps Collection Containers.



Model #50005: For use with 1-Gallon, 2-gallon and 3-gallon Sharps Collection Containers.



Model #50006: For use with 1-Quart Sharps Collection Container only.



Model #50014: For use with 1-Gallon, 2-gallon and 3-gallon Sharps Collection Containers.