20-Gallon TakeAway Recovery System^D or 30-Gallon TakeAway Recovery System^D

SYSTEM COMPONENTS (Save for packaging and return shipping.)

- Prepaid shipping box
- Inner box and bag
- Red 20-gallon (20-g) or 30-gallon (30-g) container and bag
- Tape, 2 tiewraps, absorbent pads
- Tracking form
- Instructions

INSTRUCTIONS (read instructions including all warnings completely before using this system.)

Fig. 1: System will arrive in the same box it will be shipped back in. DO NOT REMOVE BAGS. DO NOT DISCARD BOX.

Fig. 2: Leave 20-g or 30-g in box. Place in dry area. Pull outer bag over box Laps. DO NOT REMOVE ABSORBENT PADS.

Fig. 3: Place sharps containers (NO LOOSE SHARPS) and used healthcare materials in 20-g or 30-g. DO NOT OVERFILL.

Fig. 4: When full, use the tiewrap to close inner bag around materials.

Fig. 5: Close lid and latch both clasps.

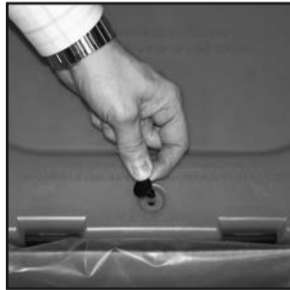
Fig. 6: Remove locking pin from top of lid and place in hole. THE LID IS NOW PERMANENTLY CLOSED.

Fig. 7: Pull second bag up around the 20-g or 30-g and close with tiewrap.

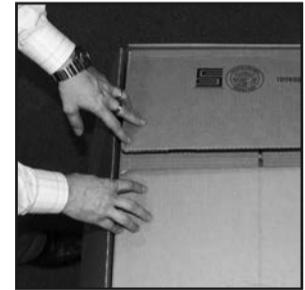
Fig. 8: Close inner box Laps.

Fig. 9: Close numbered shipping box Laps in order. Ship-to label must be showing.

Fig. 10: Tape shipping box lid closed.

Fig. 11: Remove Tracking Form from plastic pouch on shipping box. Complete top section. YOU MUST SIGN THE FORM. Remove bottom copy and keep. Place remaining copies back in pouch with barcode facing out. Box will not be accepted without tracking form.

Fig. 12: Give box to UPS. NO ADDITIONAL FREIGHT IS NECESSARY.

FOR ELECTRONIC PROOF OF TREATMENT:

To access your free SharpsTracer^D account or register as a new user go to www.sharpstracer.com. Download Proof of Treatment in 30-45 days.

ADDITIONAL WARNINGS:

- Maximum return weight including pac, aging should not exceed 50 lbs.
- (EET A) from unauthorized persons
- NO mercury or lead-containing devices or batteries in this container
- NO more than 50ml (1.66 ounces) of liquids in any sharps container
- NO drugs or trace chemo in this container unless program previously established (Call 800.772.5657 for details)
- Comply with all laws and regulations. New Jersey regulated generators attach NJ Tracking Form to Sharps Tracking Form

Call 1.800.772.5657 with emergency, leakage, damage, for replacement components, or with questions. Visit us online at www.sharpsinc.com

**PcklYfVk XY OYWmiY3/Wcsg NV3/ leYnV3 XY 0.
s 1. GVehgYk**

PUPMEMCMNMLELMP (Save for packaging and return shipping.)

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