**1-Quart & 1.5 Quart TakeAway Recovery System™ Multi-Packs**

**SYSTEM COMPONENTS (Save for packaging and return shipping.)**


- Prepaid return shipping box with pre-attached UPS shipping label (return freight is prepaid)
- Inner box
- Red sharps collection containers in individual plastic bags with absorbent pads inside
- "FULL" labels for each container
- Tape, tiewraps
- Tracking document on outside of shipping box
- Instructions

**INSTRUCTIONS (Read instructions including all warnings completely before using this system.)**


**Fig. 1:** Remove each container from packaging as needed and place at point of use. DO NOT REMOVE ABSORBENT PADS.

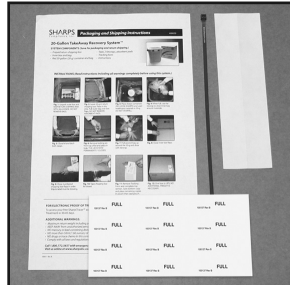**Fig. 2:** Save the shipping box and all components for repackaging the containers for return shipping.

**Fig. 3:** Place needles, syringes, lancets, other sharps and used healthcare materials into containers. DO NOT OVERFILL.

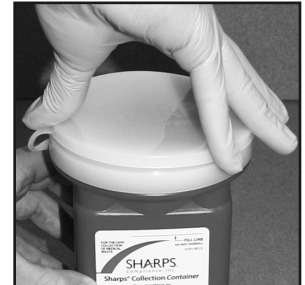**Fig. 4:** Close lid by snapping firmly over opening. Make sure lid is tightly sealed prior to packaging.


**Fig. 5:** Place "FULL" label on the top of each container.


**Fig. 6:** Once individual containers are full, securely seal the bag around each container using the tiewrap.


**Fig. 7:** Only ship shipping box when all sharps containers with which the shipping box came are enclosed.

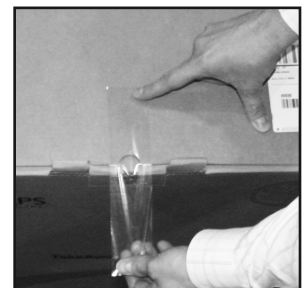**Fig. 8:** Close numbered shipping box flaps in order. Ship-to label must be showing. Tape lid closed.

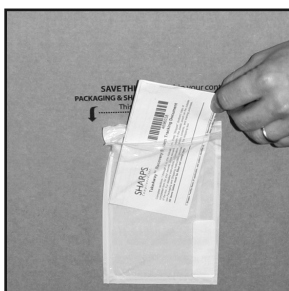
**Fig. 9:** Remove Tracking Form from plastic pouch on shipping box. Complete top section. **YOU MUST SIGN THE FORM.** Remove bottom copy and keep. Place remaining copies back in pouch with barcode facing out. Box will not be accepted without tracking form.

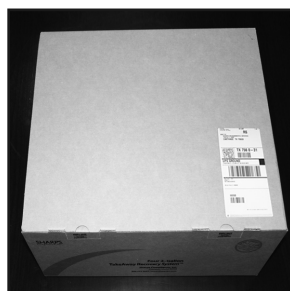**Fig. 10:** Give box to UPS. NO ADDITIONAL FREIGHT IS NECESSARY. TOTAL PACKAGE SHOULD NOT WEIGH OVER 25 LBS.

**Fig. 11:** For tracking purposes, log on to your SharpsTracer® account at [www.sharpsinc.com](http://www.sharpsinc.com). Enter Tracking Form number. Download proof of recovery in 30-45 days.

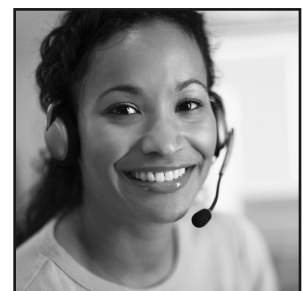**Fig. 12:** If you do not have a SharpsTracer® account, sign up for this free service by calling 800.772.5657.

**ADDITIONAL WARNINGS:**

- Maximum return weight including packaging - 25 lbs.
- KEEP AWAY from unauthorized persons
- NO mercury or lead-containing devices or batteries in this container
- NO more than 50ml (1.66 ounces) of liquids in any sharps container
- NO drugs or trace chemo in this container unless program previously established. Call Sharps at 800-772-5657 for proper labeling for these materials.
- Comply with all laws and regulations. New Jersey regulated generators attach NJ Tracking Form to Sharps Tracking Form

**Call 1.800.772.5657 with emergency, leakage, damage, for replacement components, or with questions. Visit us online at [www.sharpsinc.com](http://www.sharpsinc.com)**

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### INSTRUCTIONS (Read instructions including all warnings completely before using this system.)

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