


Two 3.5-Gallon Pails TakeAway Recovery System™

This Sharps Compliance, Inc. TakeAway Recovery System (System) contains everything required to safely contain, package and ship sharps and other used healthcare materials for proper treatment.

Go to www.sharpsinc.com for more information.

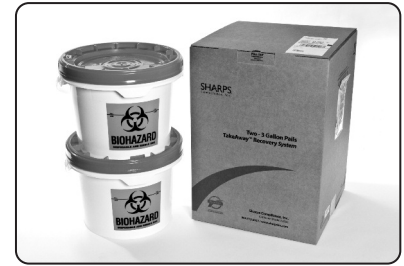

Two 3.5-Gallon Pails TakeAway™ Recovery System

CONTENTS

- Outer shipping box with pre-attached UPS shipping label (return freight is prepaid)
- Inner box lined with 4 mil plastic bag
- Two 3.5-gallon plastic pail collection containers with flanged protective inserts and absorbent pads
- Tape, twist-ties, and instructions
- Tracking Document on outside of shipping box

PRECAUTIONS – READ PRIOR TO PROCEEDING

- Do not place mercury or lead-containing devices or batteries into this container.
- Only place drugs or trace cytotoxic agents into this container if labeled for incineration. Contact Sharps at 800-772-5657 for the appropriate System or labeling method to assure proper treatment.
- Always keep pail secured from unauthorized persons.
- All persons preparing this System for shipment must follow proper packaging instruction included in this document.
- Comply with all laws and regulations regarding handling, transport, and tracking of this product. New Jersey regulated generators attach NJ Tracking Form to Sharps manifest tracking form before placing into the plastic pouch for shipping.
- Do not remove absorbent pads from plastic container(s).
- Do not stack over two containers high.
- DO NOT OVERFILL, COMPRESS, OR FORCE MATERIALS INTO THE CONTAINER(S). LID MUST FIT DOWN TIGHTLY.
- Containers are puncture-resistant, not puncture-proof.
- DO NOT ship containers in any box other than the one supplied with this system.
- Ready-to-ship package should not exceed 50 lbs.
- Empty all products free of liquids as much as possible.
- DO NOT exceed 50ml, (1.66 ounces) of liquids in container

FOR ASSISTANCE OR TO ORDER REPLACEMENT COMPONENTS, CALL 800.772.5657

TRAINING & INSTRUCTIONS FOR YOUR TAKEAWAY RECOVERY SYSTEM

Order your new TakeAway Recovery System before this system is filled.

1. Remove pail(s) from packaging and place at point of use. Save all components for repackaging System for shipping.
2. Press latch and twist counter-clockwise to remove lid (Fig. 1). Containers can be stacked.
3. To secure the protective flanged insert inside the pail, seat the notches on the insert onto the projections on the inside of the pail and twist. Keep lid on pail(s) when not in use.
4. Place used syringes and other items into pail (Fig. 2). If removing needle from syringe, i.e. clean room setting, insert needle into slot. Advance to narrow end. Twist syringe. Move syringe to large end. Needle will drop off (Fig. 3).
5. To secure lid on pail, seat lid at locations of threads on pail and twist clockwise. Make sure lid is tightly sealed.
6. If stacking pails, place full container on bottom.
7. Once full, place the pails into the plastic bag and box. Securely seal bag around pails using twist-tie.
8. Remove the Tracking Form from the plastic pouch on the side of the shipping box, and complete sections 1 through 6. Keep "Shipper" (bottom) copy of completed form for your records. Place remaining copies of the Tracking Form back in the plastic pouch on the outside of the box and seal.
9. Close the box flaps and tape.
10. For tracking purposes, log on to your SharpsTracer account at www.sharpstracer.com and enter the box number. To sign up for this free service, call 800.772.5657.
11. Either hand the box to a UPS driver, take to a UPS store, or schedule a pick-up by calling 800-772-5657. **NO ADDITIONAL FREIGHT IS NECESSARY.**
12. Upon arrival at Sharps' treatment facility, the box may be re-purposed prior to transfer for industrial use. For more information, go to www.sharpsinc.com
13. Download your Proof of Recovery from your SharpsTracer account at www.sharpstracer.com in 30-45 days.

IN CASE OF EMERGENCY OR DISCOVERY OF DAMAGE OR LEAKAGE, CALL 1.800.772.5657

For more information regarding the intended use of the Sharps collection container, call 800.772.5657.



Figure 1

Press latch and twist lid counter-clockwise to open; clockwise to close.



Figure 2

Place items into pail.



Figure 3

Proper needle removal.