
5 Gallon Medical Professional TakeAway Recovery System™

This Sharps Compliance, Inc. TakeAway Recovery System (System) contains everything required to safely contain, package and ship your sharps containers and other biohazard-contaminated items for proper treatment.

Go to www.sharpsinc.com for more information.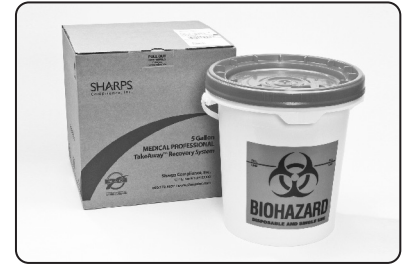

5 Gallon Medical Professional TakeAway Recovery System

CONTENTS

- Outer shipping box with pre-attached UPS shipping label (return freight is prepaid)
- Inner box lined with 4 mil plastic bag
- 5-gallon plastic pail collection container with 4 mil plastic bag liner and absorbent pads
- Tape, twist-ties, and instructions
- Tracking Document on outside of shipping box

PRECAUTIONS – READ PRIOR TO PROCEEDING

- Do not place mercury or lead-containing devices or batteries into this container
- Do not place drugs or chemotherapeutic agents into this container. Contact Sharps at 800-772-5657 for the appropriate System or labeling method to assure proper treatment.
- DO NOT PLACE LOOSE SHARPS INTO THIS PAIL
- Always keep containers secured from unauthorized persons
- All persons preparing this System for shipment must follow proper packaging instruction included in this document
- Comply with all state and federal laws and regulations regarding handling, storage, transport, and tracking of this product.
- Empty all products free of liquids as much as possible
- Do not remove absorbent pads from pail container
- DO NOT OVERFILL, COMPRESS, OR FORCE MATERIALS INTO THE CONTAINER(S). LID MUST FIT DOWN TIGHTLY.
- Ready-to-ship package should not exceed 50 lbs

FOR ASSISTANCE OR TO ORDER REPLACEMENT COMPONENTS, CALL 800.772.5657

TRAINING & INSTRUCTIONS FOR YOUR TAKEAWAY™ RECOVERY SYSTEM

Order your new TakeAway™ Recovery System before this system is filled.

1. Remove pail from packaging and place in desired location. Save all components for repackaging System for shipping.
2. Press latch and twist counter-clockwise to remove lid.
3. Place non-sharps items and sharps containers into plastic-lined 5-gallon pail. Do not place loose sharps in the pail.
4. When ready for shipping, securely seal bag around contents using twist tie.
5. To secure lid on pail, seat at locations of threads and twist clockwise. Make sure lid is tightly sealed.
6. Once full, place the pail into the plastic bag and box. Securely seal bag around pail using twist-tie.
7. Remove the Tracking Form from the plastic pouch on the side of the shipping box, and complete sections 1 through 6. Keep "Shipper" (bottom) copy of completed form for your records. Place remaining copies of the Tracking Form back in the plastic pouch on the outside of the box and seal.
8. Close the box flaps and tape.
9. For tracking purposes, log on to your SharpsTracer™ account at www.sharpstracer.com and enter the box number. To sign up for this free service, call 800.772.5657.
10. Either hand the box to a UPS driver, take to a UPS store, schedule a UPS pick-up by accessing your SharpsTracer™ account, or call 800-772-5657. NO ADDITIONAL FREIGHT IS NECESSARY.
11. Upon arrival at Sharps' treatment facility, the box will be re-purposed prior to transfer for industrial use. For more information, go to www.sharpsinc.com
12. Download your Proof of Recovery from your SharpsTracer account at www.sharpstracer.com in 30-45 days.

IN CASE OF EMERGENCY OR DISCOVERY OF DAMAGE OR LEAKAGE, CALL 1.800.772.5657

For more information regarding the intended use of the Sharps Collection Container, call 800.772.5657.

Figure 1

Press latch and twist lid counter-clockwise to open; clockwise to close.

Figure 2

Place sharps containers and non-sharps materials into the pail. NO LOOSE NEEDLES!

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