

20-Gallon Sharps Recovery System™

SYSTEM COMPONENTS (Save for packaging and return shipping.)

- Prepaid shipping box
- Inner box and bag
- Red 20-gallon (20-g) container and bag
- Tape, 2 tiewraps, absorbent pads
- Tracking form
- Instructions



INSTRUCTIONS (Read instructions including all warnings completely before using this system.)



Fig. 1: System will arrive in the same box it will be shipped back in. DO NOT REMOVE BAGS. DO NOT DISCARD BOX.



Fig. 2: You may leave the 20-g in the box for easy handling. Place in dry area, up off the floor.



Fig. 3: Place sharps containers (NO LOOSE SHARPS) and red bagged waste in 20-g. DO NOT OVERFILL.



Fig. 4: When full, use the tiewrap to close inner bag around materials.



Fig. 5: Close lid and latch both clasps.



Fig. 6: Remove locking pin from top of lid and place in hole. THE LID IS NOW PERMANENTLY CLOSED.



Fig. 7: Pull second bag up around the 20-g and close with tiewrap.



Fig. 8: Close inner box flaps.

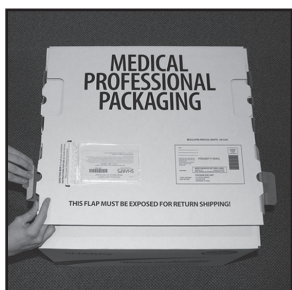


Fig. 9: Close numbered shipping box flaps in order. Disposal facility ship-to label must be showing.

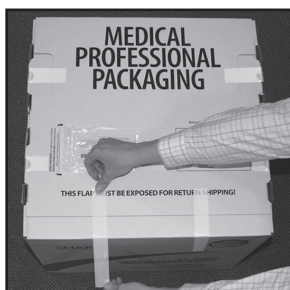


Fig. 10: Place included tape over each box flap and across the box edge as indicated.

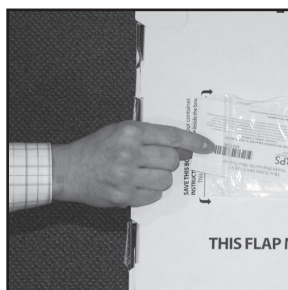


Fig. 11: Remove Tracking Form from plastic pouch on shipping box. Complete top section. YOU MUST SIGN THE FORM. Remove bottom copy and keep. Place remaining copies back in pouch with barcode facing out. Box will not be accepted without tracking form.

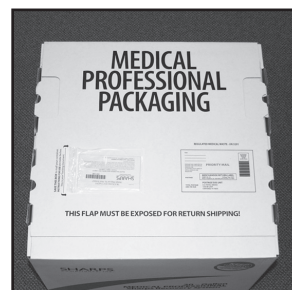


Fig. 12: Write the facility name and address in the return address section on the ship-to label. Give box to USPS driver, take to a post office, or schedule a Priority pick-up at www.sharpsinc.com. NO ADDITIONAL POSTAGE IS NECESSARY.

FOR ELECTRONIC PROOF OF TREATMENT:

To access your free SharpsTracer™ account or register as a new user go to www.sharpstracer.com. Download Proof of Treatment in 30-45 days.

ADDITIONAL WARNINGS:

- Maximum return weight including packaging should not exceed 55 lbs.
- KEEP AWAY from unauthorized persons
- NO mercury or lead-containing devices or batteries in this container
- NO more than 50ml (1.66 ounces) of liquids in any sharps container
- NO drugs or trace chemo in this container unless program previously established (Call 800.772.5657 for details)
- Comply with all laws and regulations. New Jersey regulated generators attach NJ Tracking Form to Sharps Tracking Form

Call 1.800.772.5657 with emergency, leakage, damage, for replacement components, or with questions.
Visit us online at www.sharpsinc.com