

1 & 2-Gallon TakeAway Recovery System™ Multi-Packs

SYSTEM COMPONENTS (Save for packaging and return shipping.)

- Prepaid return shipping box with pre-attached UPS shipping label (return freight is prepaid)
- Inner box and 4 mil plastic bag liner
- Red sharps collection containers with absorbent pads inside
- Tape, tiewraps
- Tracking document on outside of shipping box
- Instructions



INSTRUCTIONS (Read instructions including all warnings completely before using this system.)



Fig. 1: Remove each container from packaging as needed and place at point of use. DO NOT REMOVE ABSORBENT PADS.



Fig. 2: Save the shipping box and all components for repackaging the containers for return shipping.



Fig. 3: Place needles, syringes, lancets, other sharps and used healthcare materials into containers. DO NOT OVERFILL.



Fig. 4: Close lid by snapping firmly over opening. Make sure lid is tightly sealed prior to packaging.



Fig. 5: Only ship shipping box when all sharps containers with which the shipping box came are enclosed.



Fig. 6: When ready for shipping, securely seal bag around the containers using tiewrap.



Fig. 7: Close numbered shipping box flaps in order. Ship-to label must be showing.

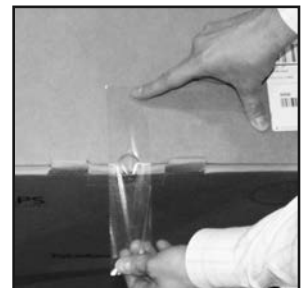


Fig. 8: Tape lid closed.



Fig. 9: Remove Tracking Form from plastic pouch on shipping box. Complete top section. **YOU MUST SIGN THE FORM.** Remove bottom copy and keep. Place remaining copies back in pouch with barcode facing out. Box will not be accepted without tracking form.



Fig. 10: Give box to UPS. NO ADDITIONAL FREIGHT IS NECESSARY.



Fig. 11: For tracking purposes, log on to your SharpsTracer® account at www.sharpsinc.com. Enter Tracking Form number. Download proof of recovery in 30-45 days.

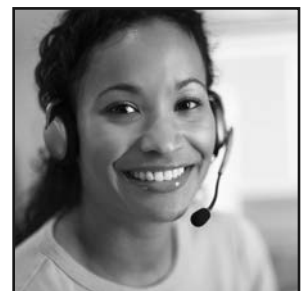


Fig. 12: If you do not have a SharpsTracer® account, sign up for this free service by calling 800.772.5657.

ADDITIONAL WARNINGS:

- KEEP AWAY from unauthorized persons
- Do not ship containers in any other box than the one provided
- NO mercury or lead-containing devices or batteries in this container
- NO more than 50ml (1.66 ounces) of liquids in any sharps container
- NO drugs or trace chemo in this container unless program previously established. Call Sharps at 800-772-5657 for proper labeling for these materials.
- Comply with all laws and regulations. New Jersey regulated generators attach NJ Tracking Form to Sharps Tracking Form

Call 1.800.772.5657 with emergency, leakage, damage, for replacement components, or with questions. Visit us online at www.sharpsinc.com